

PREPARE AND LOAD THE FLOCCARE® INFINITY™ GIVING SET INTO THE INFINITY™ III PUMP

1 Prepare the tube feed before connecting



Allow some time until
the air (foam) has
disappeared.

2 Load the Infinity™ giving set



1. Position carefully the
purple looped section
around the pump rotor.



2. Gently stretch the giving
set into position. Press
the cassette **downwards**
into the feeding pump.

3 Firmly close the door



Check if the set is properly
seated before closing.
**FIRMLY CLOSE THE
DOOR ON BOTH SIDES.**

4 Fill the Infinity™ giving set



Press and hold the
FILL SET button for
two seconds to purge
air from the giving set.

IN CASE OF AIR-ALARM:



1



SNOOZE or ACKNOWLEDGE the alarm.

2



Click on the information button to get more details on the alarm condition and resolving.

A. Check if the feeding reservoir is empty and replace as needed.

B. If feeding reservoir is not empty and air alarm is recurring:

- Open the door, press the cassette downwards to ensure it is fully seated, close the door firmly on both sides.
- If the alarm is still recurring, change the giving set.

3



Press the home button after the recommended actions have been performed.

4



(Re)start feeding.



Multilingual training website to learn more about the Infinity III pump:



ID # 400062_A
June 2024