

Sage Untreated Suction Toothbrush

With Sage Oral Solution

REF 6977-BP



Features and benefits

Suction Toothbrush

Ultra-soft nylon bristles	• Gently removes dental plaque, debris and oral secretions known to harbor respiratory pathogens
Over 3,100 bristles or “contact points”	• Cleans and removes plaque
Three suction ports	• Avoids clogging and keeps open path for debris
Available with suction handle	• Provides variable suction control and allows quick tool changes
Connects directly to standard suction lines	• Helps provide maximum suction strength
Angled handles	• Provides cleansing of the back of the oral cavity

Specifications

Product name	Sage Untreated Suction Toothbrush with Sage Oral Solution
Product code	6977-BP
Description	Single-use suction system for patients at risk for aspiration pneumonia
Manufacturer accreditation	ISO 13485:2016
Expiration date	24 months from date of manufacture
Length of Suction Toothbrush	18.4 cm
Length of each bristle	8 mm
Length of Applicator Swab	15.9 cm
Amount of solution in each burst packet	7 mL
Case quantity	100

Sage Oral Solution

- Water (aqua)
- Alcohol
- Glycerin
- PEG-40 sorbitan diisostearate
- Flavour (aroma)
- Chlorhexidine digluconate
- Sodium saccharin
- Blue 1 (CI 42090)

Packaging Information

Contents:

- (1) Untreated Suction Toothbrush, 7mL Sage Oral Solution, (1) Applicator Swab.

Indications of Use:

Aids in the removal of dental plaque, debris, and oral secretions.

Instructions for Use:

Burst solution packet with thumbs. Peel package open. Attach Suction Toothbrush to suction line. Brush teeth and oral cavity for 1 minute. To suction, place thumb over port. Rinse to clear tubing after use. Discard. Use Applicator Swab if additional cleansing is needed.

Sage Oral Solution: Do not rinse with water or other mouthwashes, brush teeth, or eat immediately after using Sage Oral Solution. Do not swallow. To minimize discolouration of teeth, brush and floss daily. STORE ABOVE FREEZING (0°C or 32°F).

Caution:

Do not allow patient to bite down on oral care tool. Use a bite block if patient has altered levels of consciousness or cannot comprehend commands. Use caution with children and unresponsive individuals. Failure to follow these safety precautions may damage the device and present a choking/aspiration hazard. May not function as intended/potential risk of cross-contamination if device is reused.

Not made with natural rubber latex.

Suction Toothbrush and Applicator Swab: MDD 93/42/EEC
Sage Oral Solution: EC Cosmetic Regulation 1223/2009

Stryker Australia Pty Ltd
 8 Herbert Street
 St Leonards NSW 2065
 61 2 9467 1000
 www.stryker.com.au

Stryker New Zealand Limited
 PO Box 17-136, Greenlane
 Auckland 1546, New Zealand
 64 9 573 1890
 www.stryker.com

A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that healthcare professionals be trained in the use of any particular product before using it. The information presented is intended to demonstrate the breadth of Stryker product offerings. A healthcare professional must always refer to the package insert, product label and/or instructions for use before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area. Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker. All other trademarks are trademarks of their respective owners or holders.

For more information please contact your local Stryker sales representative.