

MiniONE[®] Button


Low Profile Balloon Gastrostomy Tube



Reduce Leaks &
Improve Comfort

Improve Patient
Outcomes

Easy Nighttime use
Glow in the Dark

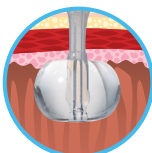

Innovating. Educating. Changing Lives.

What Makes A Better Balloon Button?

Reduce Leaks

The AMT MiniONE® is designed to maximise seal at the gastric surface for a better fit.

- **Apple Shaped Balloon** grows horizontally when filled, covering more surface area at the gastrostomy site.
- **Tapered Transition** from the shaft to the base of the external bolster is designed for a more secure fit.
- **Durable** anti-reflux valve offers maximum protection.
- **Adjustable** balloon allows for fill variation to ensure individualised proper fit.



Balloon Fill Volumes

French Size	Minimum	Recommended	Maximum
12F	2 ml	2.5 ml	3 ml
14F	3 ml	4 ml	5 ml
16F	4 ml	6 ml	8 ml
18F	6 ml	8 ml	10 ml
20F	7 ml	10 ml	15 ml
24F	7 ml	10 ml	15 ml

Consult your healthcare provider before adjusting fill volumes.

Recommended amount of water to fill the balloon is printed on fill valve




Improve Comfort

Maximization of comfort is key when it comes to patient, caregiver and clinician satisfaction with an enteral feeding device.

- **Low Profile** external bolster lays discreetly beneath clothing.
- **Low Profile** internal retention balloon occupies less gastric space.
- **Soft** flexible high grade silicone is friendly to skin surface and easy to clean around.
- **Unobtrusive Fill Port** angled away from the body for reduced irritation to skin.
- **Glow Green** feeding port and feed set connector glows in the dark to assist caregivers with administration of nighttime feeds avoiding the need to wake the patient.



Clearly Labeled

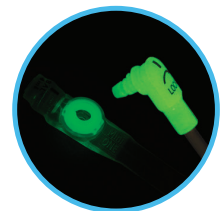

Glow Green for Nighttime Use




Softer


*Lower than the leading competitor

The AMT MiniONE® is designed to best fulfill a patient's nutritional needs with as little lifestyle disruption as possible.

Low Profile External Bolster

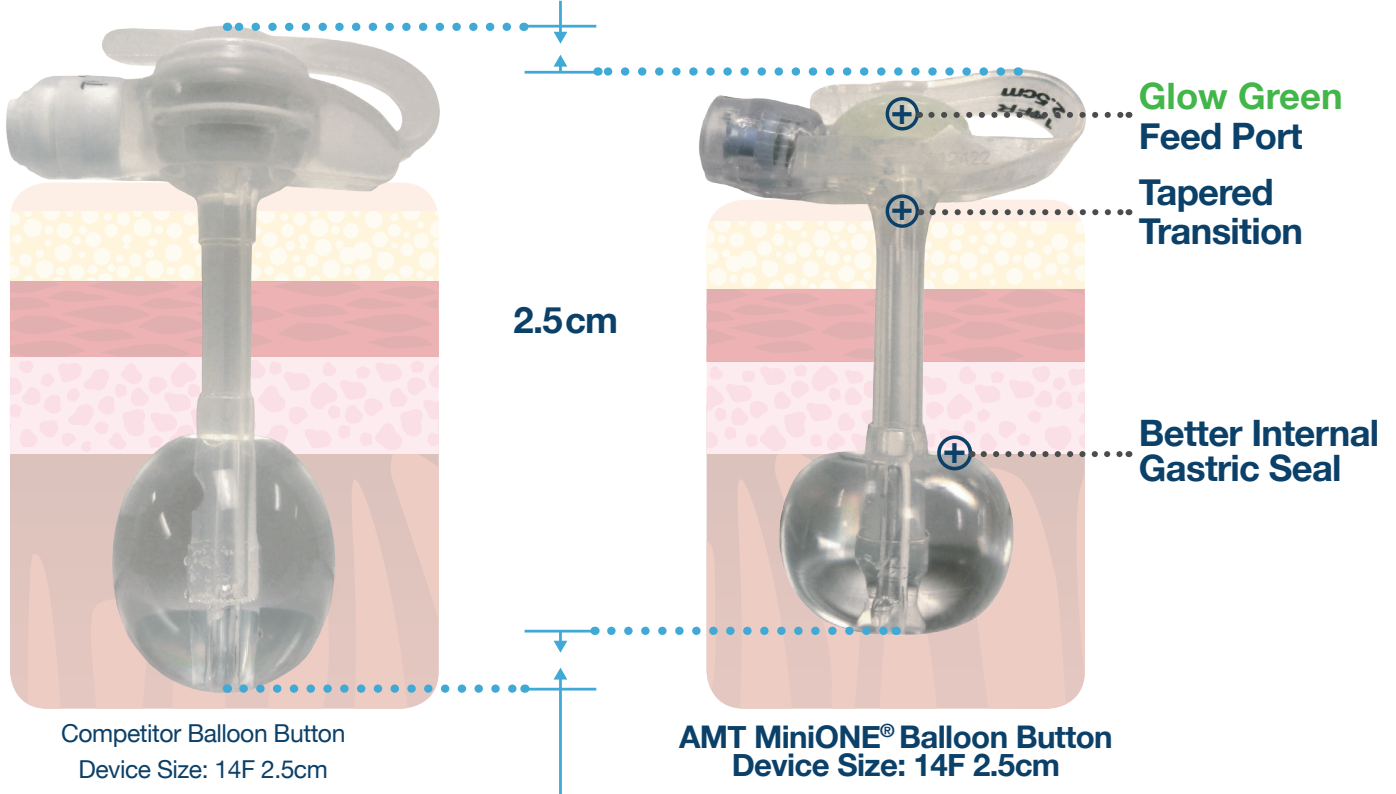

Low Profile Within The Stomach

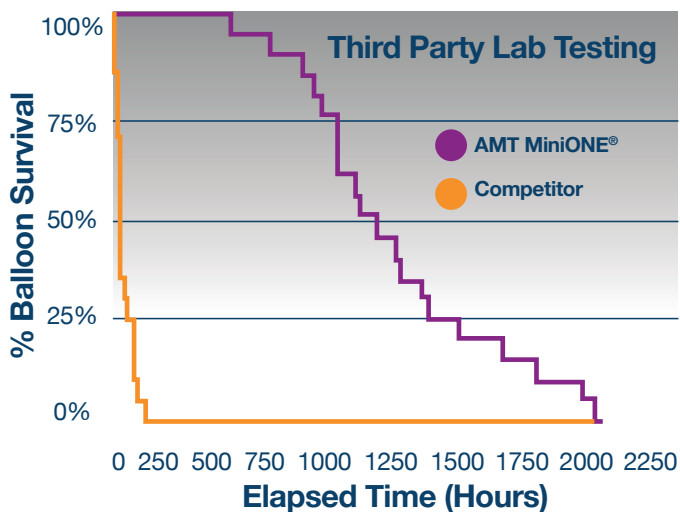

For Maximum Balloon Life

It is crucial to measure stoma length using the AMT Balloon Stoma Measuring Device for proper fit.




Non-Guidewire Compatible
Order #: A2-1704

AMT Balloon Stoma Measuring Device

Proper Fit Every Time.

It is crucial to measure stoma length using the AMT Balloon Stoma Measuring Device for proper fit, especially when a patient switches to AMT from another brand.

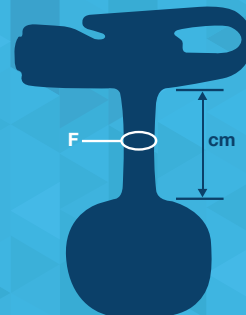

F = Tube Diameter
cm = Stoma Length

Example: 14F x 2.3 cm

(F) French Size
(cm) Centimeter

Balloon Stoma Measuring Device	Order #	Packaged
Non-Guidewire Compatible	A2-1704	10/Box

MiniONE® Balloon Button Sizes

12 F		14 F		16 F		18 F		20 F		24 F	
(cm) Length	Order #	(cm) Length	Order #	(cm) Length	Order #	(cm) Length	Order #	(cm) Length	Order #	(cm) Length	Order #
0.8	A2-M1-51208-I	0.8	A2-M1-51408-I								
1.0	A2-M1-51210-I	1.0	A2-M1-51410-I	1.0	A2-M1-51610-I	1.0	A2-M1-51810-I	1.0	A2-M1-52010-I		
1.2	A2-M1-51212-I	1.2	A2-M1-51412-I	1.2	A2-M1-51612-I	1.2	A2-M1-51812-I	1.2	A2-M1-52012-I		
1.5	A2-M1-51215-I	1.5	A2-M1-51415-I	1.5	A2-M1-51615-I	1.5	A2-M1-51815-I	1.5	A2-M1-52015-I	1.5	A2-M1-52415-I
1.7	A2-M1-51217-I	1.7	A2-M1-51417-I	1.7	A2-M1-51617-I	1.7	A2-M1-51817-I	1.7	A2-M1-52017-I	1.7	A2-M1-52417-I
2.0	A2-M1-51220-I	2.0	A2-M1-51420-I	2.0	A2-M1-51620-I	2.0	A2-M1-51820-I	2.0	A2-M1-52020-I	2.0	A2-M1-52420-I
2.3	A2-M1-51223-I	2.3	A2-M1-51423-I	2.3	A2-M1-51623-I	2.3	A2-M1-51823-I	2.3	A2-M1-52023-I	2.3	A2-M1-52423-I
2.5	A2-M1-51225-I	2.5	A2-M1-51425-I	2.5	A2-M1-51625-I	2.5	A2-M1-51825-I	2.5	A2-M1-52025-I	2.5	A2-M1-52425-I
2.7	A2-M1-51227-I	2.7	A2-M1-51427-I	2.7	A2-M1-51627-I	2.7	A2-M1-51827-I	2.7	A2-M1-52027-I	2.7	A2-M1-52427-I
3.0	A2-M1-51230-I	3.0	A2-M1-51430-I	3.0	A2-M1-51630-I	3.0	A2-M1-51830-I	3.0	A2-M1-52030-I	3.0	A2-M1-52430-I
3.5	A2-M1-51235-I	3.5	A2-M1-51435-I	3.5	A2-M1-51635-I	3.5	A2-M1-51835-I	3.5	A2-M1-52035-I	3.5	A2-M1-52435-I
4.0	A2-M1-51240-I	4.0	A2-M1-51440-I	4.0	A2-M1-51640-I	4.0	A2-M1-51840-I	4.0	A2-M1-52040-I	4.0	A2-M1-52440-I
		4.4	A2-M1-51444-I	4.4	A2-M1-51644-I	4.4	A2-M1-51844-I	4.4	A2-M1-52044-I	4.4	A2-M1-52444-I
		5.0	A2-M1-51450-I			5.0	A2-M1-51850-I				
		5.5	A2-M1-51455-I								
		6.0	A2-M1-51460-I			6.0	A2-M1-51860-I				
		6.5	A2-M1-51465-I								

** All MiniONE kits above are packaged with one A2-81255-ISOSAF Y-Port feed set and one A2-81211-ISOSAF Bolus feed set.

Compatible Feed Sets

		Connector Options					
		Right Angle			Straight		
		2"	12"	24"	2"	12"	24"
Adapter Options	Y-Port	A2-80222-ISOSAF	A2-81255-ISOSAF**	A2-82455-ISOSAF	A2-80212-ISOSAF	A2-81212-ISOSAF	A2-82455-ISOSAF
	Bolus	A2-80211-ISOSAF	A2-81221-ISOSAF	A2-82421-ISOSAF	A2-80211-ISOSAF	A2-81211-ISOSAF**	A2-82411-ISOSAF

All MiniONE® feed sets are compatible with Mic-Key® devices.

Note: Feeding sets are supplied assembled. Choose the best combination of length, connector and adapter type for your use.