

Atrauman® Ag

Silver Wound Contact Layer




Impregnated Silver Wound Contact Layer from HARTMANN.

Atrauman® Ag is a low adherent wound contact layer which contains silver. Designed specifically for contaminated, traumatic wounds helping reduce the risk of infection.

Reduces the Risk of Bacterial Burden.

Atrauman® Ag is coated in silver which has been proven to minimise the bacterial burden on wounds.^[1] The silver ions bind with bacteria and destroy them within the dressing whilst protecting the wound bed.

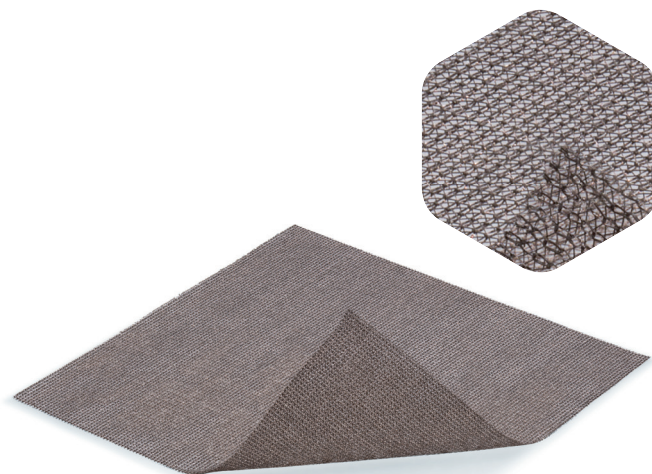

FOR A WIDE RANGE OF INDICATIONS



Skin Tears




Infected Post Op Wounds




Grazes



Diabetic Foot Ulcers

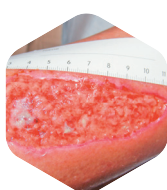


Arterial Ulcers

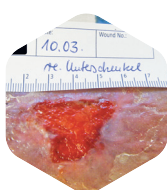

Venous Leg Ulcers

APPLICATION

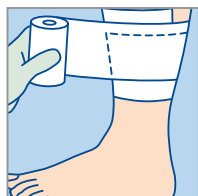

BENEFITS

- Effectively kills a broad spectrum of bacteria
- Lowers toxicity to human cells thanks to reduced release of silver ions ^[1]
- Fine weave allows passage of exudate away from the wound surface
- Improved wound healing outcomes
- Infection prevention in wound treatment

SUGGESTED WEAR TIME

- Up to 3 days

SUGGESTED SECONDARY DRESSINGS

- Zetuvit® Plus Silicone - for moderately exuding wounds
- Zetuvit® Plus - for highly exuding wounds

SUGGESTED FIXATION

- Peha-haft® - cohesive bandage
- Omnifix® - non woven tape
- Hydrofilm® - waterproof film

PRODUCT	SIZE	UNITS PER CASE	CODE
Atrauman® Ag <i>Sterile, individually sealed</i>	5 x 5 cm	120 pcs	499571
	10 x 10 cm	120 pcs	499573
	10 x 20 cm	60 pcs	499575

[1] K. Ziegler et al. (2006). Reduced Cellular Toxicity of a New Silver Containing Antimicrobial Dressing and Clinical Performance in Non-Healing Wounds. Skin Pharmacol Physiol 2006;19:140-146