

# Instruction for Use

**[Product name]** Safety Pen Needles

**[Model]** HS

**[Specification]** 30G×3/16" 0.30mm×5mm、30G×5/16" 0.30mm×8mm  
31G×3/16" 0.25mm×5mm、31G×1/4" 0.25mm×6mm、31G×5/16" 0.25mm×8mm  
32G×5/32" 0.23mm×4mm、32G×3/16" 0.23mm×5mm、32G×1/4" 0.23mm×6mm  
33G×5/32" 0.20mm×4mm、33G×3/16" 0.20mm×5mm

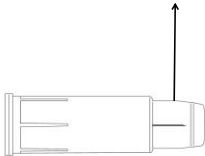
**[The period of validity]** 5 years

## **[Product Structure and Performance]**



### **Before use:**

Front shield covering  
needle

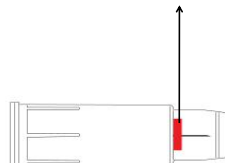


### **After use:**

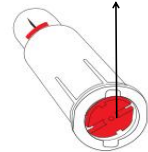
Pen connection end



Red band indicator



Red shield



## **[Intended use]**

The safety pen needle is intended for use with pen injector devices for subcutaneous injection of drugs.

Additionally, the product has two safety shields, which lock in place after use (patient end) and upon removal of the needle from the pen (pen connection end). The locked shields help reduce the occurrence of needle sticks from both ends of the needle.



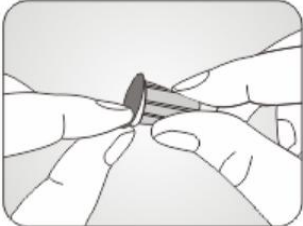
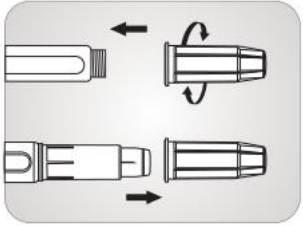
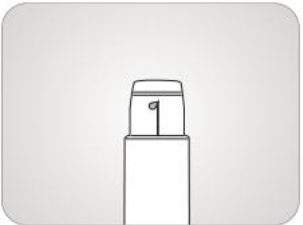

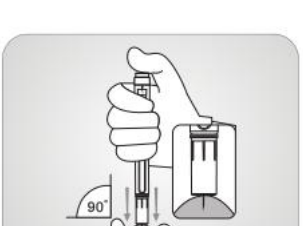
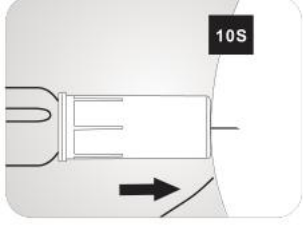
## **[Precaution and warning]**

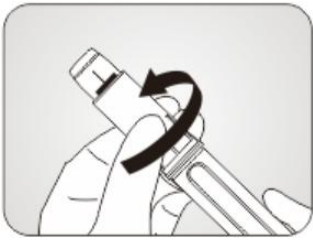
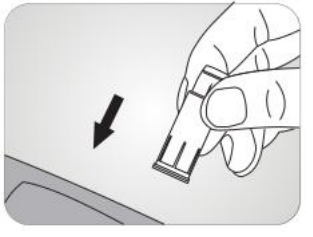
- Consult your doctor if you have any problems for use of this product.
- Select the appropriate size of needle according to the doctor's suggestion.
- Please carefully read the instruction for use of the pen injector used with the safety pen needle.
- Do not use the safety pen needle if the sealing paper is damaged.
- The period of validity of safety pen needle is 5 years. Do not use the product beyond the expiry date.
- The safety pen needle is disposable and please use a new needle before each injection.
- Before each injection, ensure the liquid could pass the needle tubing smoothly according to the Step 3 of use method.
- Discard the used needle into a sharps container according to all local regulations.
- Keep it away from the children and out of their reach.
- Please do not reuse the needle. Reuse may cause the needle to break, increase injection pain, and cannot guarantee the sterility of the needle.

- If you are stuck by a needle already used by others, please consult a doctor promptly.
- Do not re-sterilize the needle.

**[Contraindications]** No known contraindications.

**[Use method]**

	<p>1. Peel off the sealing paper.</p>
	<p>2. Holding the outer cap, vertically screw the pen needle onto the pen, and then pull the outer cap straight off.</p>
	<p>3. Always check the flow of the pen needle before each injection. Refer to the procedure described in the instruction for use of the delivery device. A drop of liquid should appear at the needle tip. If not, repeat until liquid appears at the needle tip. <b>Note:</b> Liquid may fall back into the pen needle which may result in fluid being left on the skin after injection.</p>
	<p>4. Set the dose prescribed by your doctor following the instruction for use of the pen injector. Please consult your doctor which injection method is most suitable for you. <b>4A- NO PINCH METHOD</b> Hold the pen with one hand, in palm. Your thumb should not be on the button of the delivery device. Insert the pen needle STRAIGHT into the skin in one continuous motion until the Main Body touches the skin. Maintain constant pressure against the skin (do not change hands).</p>
	<p><b>4B- PINCH METHOD</b></p> <ul style="list-style-type: none"> <li>● To avoid injecting into the muscle(e.g. young children, low body fat)</li> <li>● Or to ensure a firm skin surface for a successful injection</li> </ul> <p>Lift a 1 inch(2.5cm) skinfold (pinch up). Hold the pen with one hand, in palm. Your thumb should not be on the button of the delivery device. Insert the pen needle straight into the skin in one continuous motion until the Main Body touches the skin. Maintain constant pressure against the skin (do not change hands).</p>
	<p>5. Inject your dose following the instruction for use of the pen injector. After the injection, leave the needle in the skin for 10 seconds to ensure the dose accuracy.</p>

	<p>6. Withdraw the pen needle from the skin once the entire dose has been given and a red band indicator shows the safety shield is activated automatically.</p> <p>Hold the main body of the Pen Needle and screw off the pen needle. Pen Connection End of pen needle is now protected by the red shield.</p> <p>DO NOT place your fingers on either end of the pen needle shields.</p>
	<p>7. Discard the used Safety Pen Needle into a sharps container in accordance with all local regulations.</p>

**[Pen injectors compatible with Safety Pen Needle]**

- NovoPen® 5
- Autopen®24
- Gansulin Pen
- Ypsopen®
- UNIPEN®
- XiuLin Pen
- HumaPen®Ergoll
- Byetta
- Victoza Pen

**[Symbols and other instructions]**



Sterilized using irradiation



Do not re-use



Do not use if package is damaged



Caution



Not made with natural rubber



Medical device



Batch code



Use-by date



Consult instructions for use



Manufacturer



Non-pyrogenic

The product is sterile, single use only and nonpyrogenic.



Sandstone Medical ( Suzhou ) Inc .

Building 6, No.168 PuTuoShan Road, New District, 215153 Suzhou, Jiangsu Province, PEOPLE'S REPUBLIC OF CHINA

**【Effective date】** January 19, 2024 (Version 01)