



sutherlandmedical

Introducing **Ezy-As™** Compression Stocking / Garment Applicator:



New lines Available now

Extra Large (green)

Extra Small (grey)



COMPRESSION BANDAGES



sutherlandmedical

Ezy-As™ Compression Stocking / Garment Applicator

Designed to assist users with application of compression stockings, tubular bandages or garments

The compression stocking or tubular bandage is turned inside out over the applicator evenly distributing elastic forces of the garment onto the smooth outer surface of the device. Once the garment is correctly positioned on the applicator the user then transfers the compression garment onto the limb.

Features:

- Works with both open and closed toe stocking
- Fits compression garments to upper and lower limbs
- Significantly places less stress and pain on the injury and / or limb
- Robust, durable and easy to clean
- Compact, lightweight and travel friendly
- **Australian Made** and designed
- Available in **five** colour coded sizes for easy identification
- Easy to use and attach handles (sold separately)



Care Advice: Clean with warm water and mild detergent. If disinfection required, the applicator may be wiped with a 70% isopropyl alcohol solution. DO NOT use solvents or steam sterilisation. DO NOT store in direct sunlight or near a heat source. (Failure to adhere to these care instructions will void the lifetime guarantee)

Code	Description	Size	Colour	Calf/ Bicep Circumference	Qty
SM258	Ezy-As™ Applicator	Extra Small	Grey	15 - 20cm	1
SM259	Ezy-As™ Applicator	Small	Orange	28 - 35cm	1
SM260	Ezy-As™ Applicator	Medium	Yellow	32 - 40cm	1
SM261	Ezy-As™ Applicator	Large	Blue	38 - 47cm	1
SM262	Handles	Fits all sizes	Black		1
SM263	Ezy-As™ Applicator	Extra Large	Green	45 cm+	1

Refer to Ezy-As™ Instruction Guide for helpful tips.

Sutherland Medical Pty Limited

Unit 2, 195 Chesterville Road Moorabbin Vic 3189

Ph: 1300 664 027

Email: orders@sutherlandmedical.com.au

Web Site: www.sutherlandmedical.com.au

ARTG 164835

Version 5 / Created 2/9/21