

PBS ECO Series

The PBS ECO series of cylindrical disposable container bases are made from 100% recycled materials. The different shaped indents on the lid ease the disposal and the removal of needles, while the flaps on the lid prevent the accidental outflow of the sharp contents.

All models have an easy to use temporary and permanent closure and all are available with a carrying handle. All PBS ECO sharps disposal containers are certified compliant to sharps container standard ISO 23907-1:2019, on which the Australian/NZ standard AS/NZS 23907:2023 is based.

1. Attaching the lid



Position the lid on the container with the waste insertion opening on the marked side.



Press firmly all around the lid with the palms of your hands. Clicks will be heard with each press, indicating proper assembly.



Before use, fully press the anti-reflux fins into a funnel shape to facilitate the insertion of contaminated items. The anti-reflux device prevents objects from coming out of the collector.

2. Needle removal



Adhere to the maximum filling level indicated on the container.

3. Lid closure



Temporary closure: Press firmly on the full side closure system.



Permanent closure: Press with the palms of your hands on the closure system on the hollow side until it clicks.



Do not exceed the designated fill level on these containers



The integrated handle at the top of the container can be used for ease of transportation

Disposal Method

Dispose in accordance with facility or local protocols

Caution:

- Do not place hands in bin
- Keep container upright
- Do not empty or re-use