

Measure for JOBST® Medical LegWear

Measuring positions

cH - Hip

Circumference at largest dimension around buttocks

cH

cG - Thigh

Circumference at largest dimension

cG

cC - Calf


Circumference at largest dimension

cC

cB - Ankle

Circumference at smallest dimension

cB

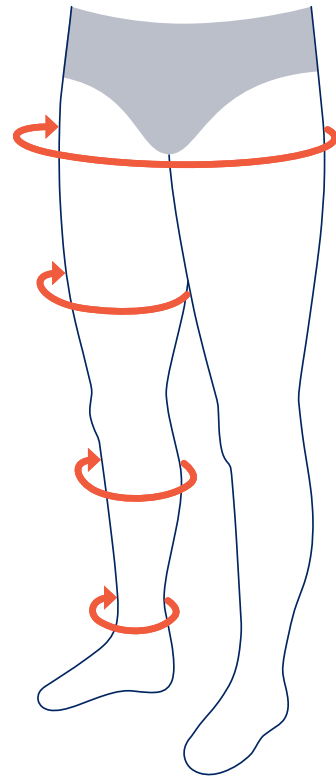

Provide the following measurements

Knee High - cB & cC

Thigh High - cB, cC & cG

Waist high - cB, cG & cH

NOTE Each leg is measured independently and measure in the early morning when possible as swelling often increases later in the day.

Style	Measurement		S	M	L	L Full Calf	XL	XL Full Calf
UNISEX	Ankle		18 - 21 cm	21 - 25 cm	25 - 29 cm	25 - 30 cm	29 - 33 cm	30 - 36 cm
	Calf		28 - 38 cm	30 - 42 cm	32 - 46 cm	46 - 61 cm	34 - 50 cm	46 - 61 cm
	Thigh		40 - 62 cm	46 - 70 cm	54 - 78 cm		60 - 81 cm	
	Hip		71 - 117 cm	76 - 127 cm	81 - 137 cm		102 - 166 cm	
MATERNITY	Ankle		18 - 21 cm	21 - 25 cm	25 - 29 cm		29 - 33 cm	
	Thigh		40 - 62 cm	46 - 70 cm	54 - 78 cm		60 - 81 cm	
	Hip		81 - 132 cm	86 - 142 cm	91 - 152 cm		112 - 180 cm	

UltraSheer: Petite Knee High Length 34 - 39 cm, Regular Knee High Length 39 - 45 cm