

# Kangaroo™ Joey Enteral Feeding Pump User Guide (Feed & Hydration Mode)



## Pump set loading procedure.

- 

1. Locate power button on lower right hand corner of control panel. Press to turn pump on.
- 

2. Confirm if you want to **KEEP** or **CLEAR** any prior pump settings.
- 

3. With pump display facing you open blue door at top of pump.
- 

4. Grasp finger tab on valve and insert valve into valve slot on the left.
- 

5. Grasp black ring retainer, and gently wrap tubing anti-clockwise around rotor. Pull up retainer and insert into slot.
- 

6. Close blue door. Display should read **SET LOADED**.

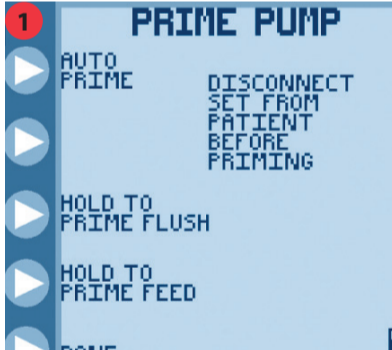
## Power down procedure

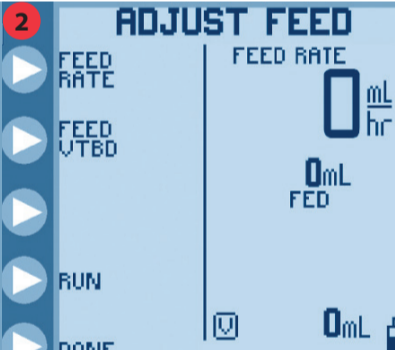
Locate power button on lower right hand corner of control panel. Press and hold power button until the pump displays turns off.

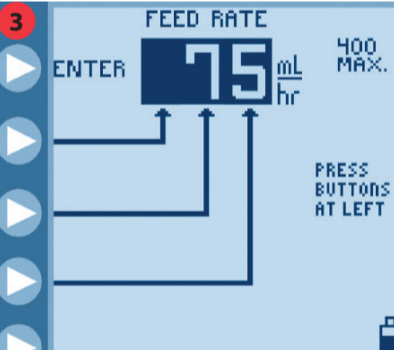
## Alarm code information.


ALARM CODE	DESCRIPTION	ACTION
<b>HOLD ERROR</b>	Pump left in Hold for more than 10 minutes.	Press <b>CONTINUE</b> followed by <b>RUN</b> to start pump running or power down pump.
<b>ROTOR ERROR</b>	Problem with pump set tubing around the rotor.	Ensure silicone tubing is correctly loaded around rotor.
<b>FLOW ERROR</b>	Obstruction in line downstream from pump.	Check for kinked or blocked tubing below pump and rectify.
<b>FEED ERROR</b>	Feed container empty or obstruction in the line upstream from pump.	Check for kinked tubing or obstruction in line above pump or attach new/refilled feed container.
<b>FLUSH ERROR</b>	Water container empty or obstruction in the line upstream from pump.	Check for kinked tubing or obstruction in flush line above pump or refill flush container.
<b>FEEDING COMPLETE</b>	Programmed feed has been delivered.	Power down pump.
<b>PUMP SET DISLODGED</b>	Black retainer ring (MISTIC) is not properly loaded in MISTIC pocket.	Ensure pump set is correctly loaded.
<b>BATTERY LOW</b>	Battery must be charged immediately.	Plug power cord into AC outlet.
<b>SYSTEM ERROR</b>	General error - see pump manual.	Send pump for service.

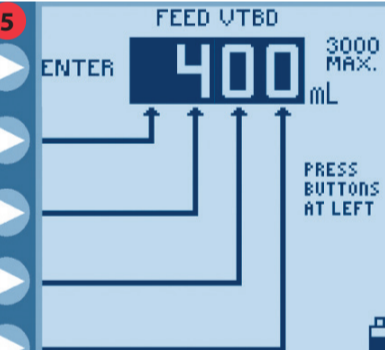
## Set priming and pump programming procedure.

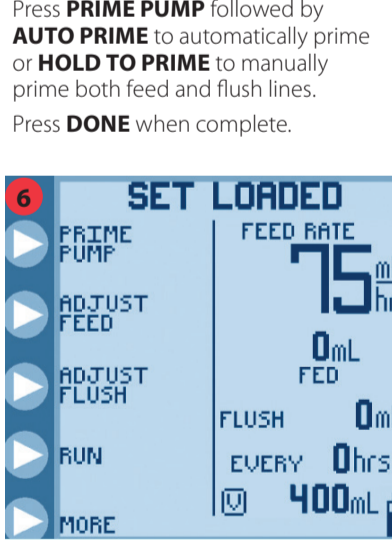
- 

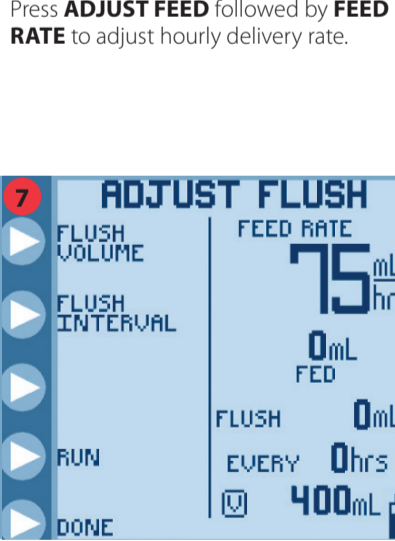
1. Press **PRIME PUMP** followed by **AUTO PRIME** to automatically prime or **HOLD TO PRIME** to manually prime both feed and flush lines. Press **DONE** when complete.
- 

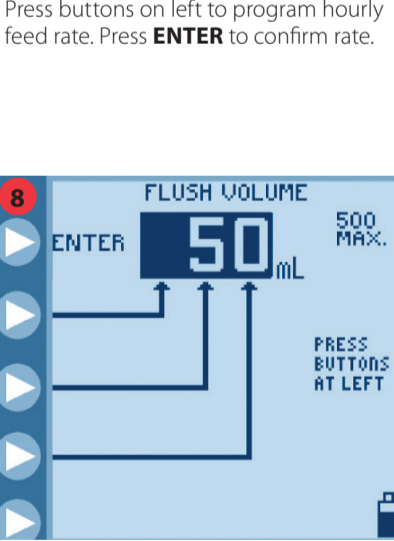
2. Press **ADJUST FEED** followed by **FEED RATE** to adjust hourly delivery rate.
- 

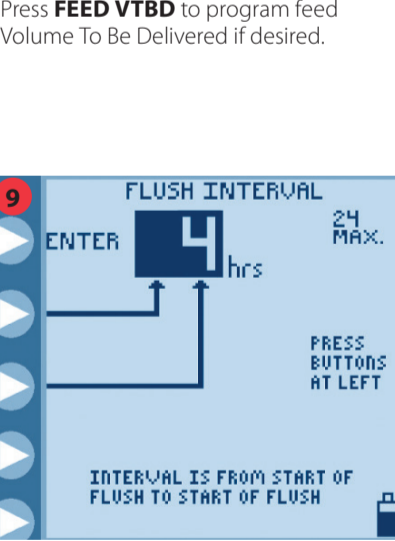
3. Press buttons on left to program hourly feed rate. Press **ENTER** to confirm rate.
- 

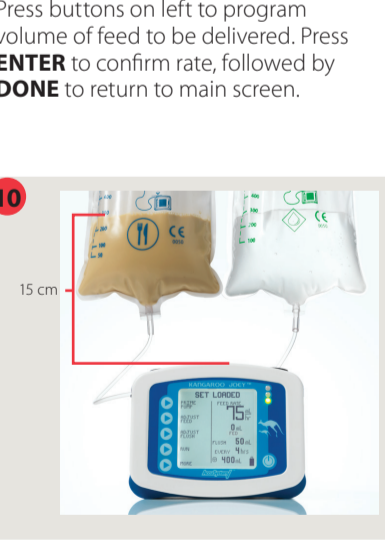
4. Press **FEED VTBD** to program feed Volume To Be Delivered if desired.
- 

5. Press buttons on left to program volume of feed to be delivered. Press **ENTER** to confirm rate, followed by **DONE** to return to main screen.
- 

6. Press **ADJUST FLUSH** to program flush volume and interval.
- 

7. Press **FLUSH VOLUME** to program flush volume. Press **FLUSH INTERVAL** to program interval between each flush.
- 

8. Press buttons on left to program volume of flush to be delivered. Press **ENTER** to confirm volume.
- 

9. Press buttons on left to program flush interval. Press **ENTER** to confirm interval, followed by **DONE** to return to main screen.
- 

10. Press **RUN** to commence feeding regime. In order to achieve proper accuracy, the initial fluid line in the feeding set container must be **15 cm** above the top of the feeding pump when initiating the feeding cycle.

**ALWAYS READ THE LABEL AND FOLLOW THE INSTRUCTIONS FOR USE. CONTACT YOUR HEALTH CARE PROFESSIONAL IF YOU HAVE ANY CONCERNS OR QUESTIONS. THIS PRODUCT IS NOT AVAILABLE FOR PURCHASE BY THE GENERAL PUBLIC.**

For complete operation information please refer to Kangaroo™ Joey enteral feeding pump operation and service manual.

**\* Important Note.** Joey Pump will automatically flush an additional 25mL of water after completing the programmed delivery of formula, as long as at least one flush had occurred during formula delivery.