

Kangaroo™ Joey Enteral Feeding Pump User Guide (Feed Only Mode)



Pump set loading procedure.

- 

1. Locate power button on lower right hand corner of control panel. Press to turn pump on.
- 

2. Confirm if you want to **KEEP** or **CLEAR** any prior pump settings.
- 

3. With pump display facing you open blue door at top of pump.
- 

4. Grasp finger tab on valve and insert valve into valve slot on the left.
- 

5. Grasp black ring retainer, and gently wrap tubing anti-clockwise around rotor. Pull up retainer and insert into slot.
- 

6. Close blue door. Display should read **SET LOADED**.

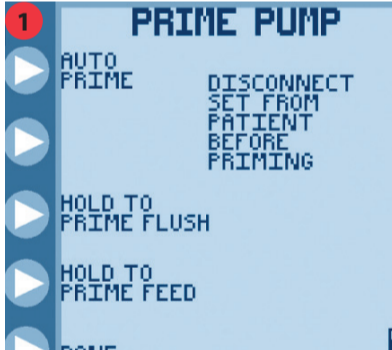
Power down procedure

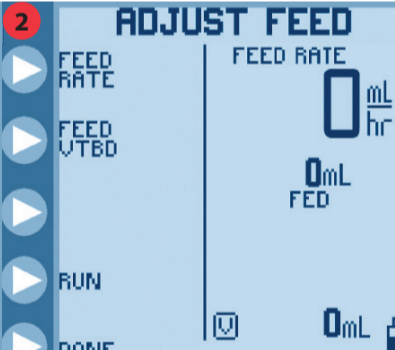
Locate power button on lower right hand corner of control panel. Press and hold power button until the pump displays turns off.

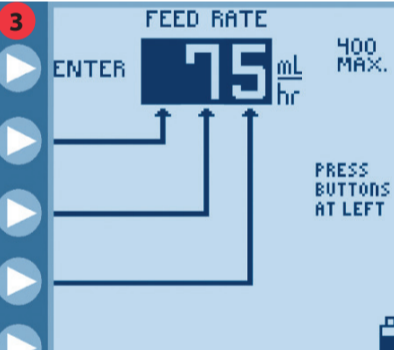
Alarm code information.


ALARM CODE	DESCRIPTION	ACTION
HOLD ERROR	Pump left in Hold for more than 10 minutes.	Press CONTINUE followed by RUN to start pump running or power down pump.
ROTOR ERROR	Problem with pump set tubing around the rotor.	Ensure silicone tubing is correctly loaded around rotor.
FLOW ERROR	Obstruction in line downstream from pump.	Check for kinked or blocked tubing below pump and rectify.
FEED ERROR	Feed container empty or obstruction in the line upstream from pump.	Check for kinked tubing or obstruction in line above pump or attach new/refilled feed container.
FLUSH ERROR	Water container empty or obstruction in the line upstream from pump.	Check for kinked tubing or obstruction in flush line above pump or refill flush container.
FEEDING COMPLETE	Programmed feed has been delivered.	Power down pump.
PUMP SET DISLODGED	Black retainer ring (MISTIC) is not properly loaded in MISTIC pocket.	Ensure pump set is correctly loaded.
BATTERY LOW	Battery must be charged immediately.	Plug power cord into AC outlet.
SYSTEM ERROR	General error - see pump manual.	Send pump for service.

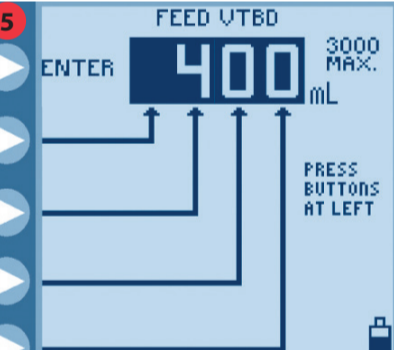
Set priming and pump programming procedure.

- 

1. Press **PRIME PUMP** followed by **AUTO PRIME** to automatically prime or **HOLD TO PRIME** to manually prime both feed and flush lines. Press **DONE** when complete.
- 

2. Press **ADJUST FEED** followed by **FEED RATE** to adjust hourly delivery rate.
- 

3. Press buttons on left to program hourly feed rate. Press **ENTER** to confirm rate.
- 

4. Press **FEED VTBD** to program feed Volume To Be Delivered if desired.
- 

5. Press buttons on left to program volume of feed to be delivered. Press **ENTER** to confirm rate.

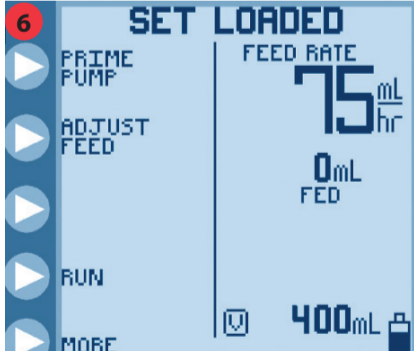
Press **PRIME PUMP** followed by **AUTO PRIME** to automatically prime or **HOLD TO PRIME** to manually prime both feed and flush lines. Press **DONE** when complete.


Press **ADJUST FEED** followed by **FEED RATE** to adjust hourly delivery rate.

Press buttons on left to program hourly feed rate. Press **ENTER** to confirm rate.

Press **FEED VTBD** to program feed Volume To Be Delivered if desired.

Press buttons on left to program volume of feed to be delivered. Press **ENTER** to confirm rate.

- 

6. Press **DONE** to return to main screen and then press **RUN** to commence feeding regime.
- 

7. In order to achieve proper accuracy, the initial fluid line in the feeding set container must be **15 cm** above the top of the feeding pump when initiating the feeding cycle.

Press **DONE** to return to main screen and then press **RUN** to commence feeding regime.

In order to achieve proper accuracy, the initial fluid line in the feeding set container must be **15 cm** above the top of the feeding pump when initiating the feeding cycle.

For complete operation information please refer to Kangaroo™ Joey enteral feeding pump operation and service manual.

Cardinal Health Australia.

Level 2, 5 Eden Park Drive, North Ryde NSW 2113. © 2021 Cardinal Health. All Rights Reserved. CARDINAL HEALTH, the Cardinal Health LOGO and ESSENTIAL TO CARE are trademarks of Cardinal Health and may be registered in the US and/or in other countries. All other marks are the property of their respective owners. 1929 ref # MSCR-81

cardinalhealth.com.au