

MiniONE[®] Button

Low Profile Balloon Gastrostomy Tube



Reduce Leaks

Improve Comfort

Soft + Flexible

Glow Green[™] Technology

Innovating Better Balloon Devices

AMT MiniONE® Balloon Button



Exclusive AMT Balloon
Secure Gastric Seal



Soft & Flexible
Optimised for Comfort



Glow Green™
Facilitates Nighttime Feeds

An Advantage in Enteral Nutrition

Reduce Leaks

The MiniONE® Balloon Button was designed to reduce gastric leakage that can cause nutrient loss, skin damage and granulation tissue

- **Exclusive AMT Balloon**
apple-shaped, expands horizontally
>50% more internal stoma coverage
- **Tapered Transition**
from bolster to tube, for a more secure fit
- **Anti-Reflux Valve**
prevents backflow
- **Adjustable Balloon Volume**
ensures proper fit

Soft + Flexible

- **Flexible External Bolster**
easy to clean around the stoma site
- **Gentle to the Skin**
designed to reduce granulation tissue
- **Balloon Fill Port**
angled away from the body to reduce skin irritation at the stoma site

Improve Comfort

AMT understands that comfort is key when it comes to patient, caregiver and clinician satisfaction

- **37% Lower External Profile¹**
MiniONE® lays discreetly below clothing
- **30% Lower Balloon Profile¹**
occupies less space within the stomach



Your ONE Source for Enteral Nutrition Devices. Accessories. Resources. Support

MiniONE® Balloon Button

37% Lower Profile¹

External Bolster

Glow Green™ Feed Port

Feed The Green

Balloon Fill Valve

Angled to Reduce Irritation

Anti-Reflux Valve

Tapered Transition

Soft + Flexible

Rigid

Competitor Device

Superior Gastric Seal

>50% More Coverage¹

Adjustable Balloon Volume


Ensures Proper Fit

30% Lower Profile¹

Within The Stomach

Simple + Secure Feed Set Attachment

To attach a feed set to a MiniONE® Balloon Button, follow these easy steps:

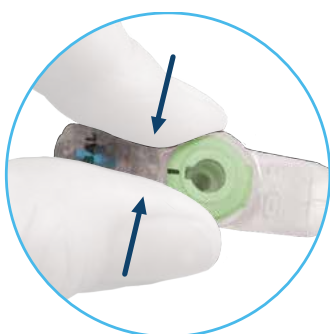

STABILIZE



ALIGN & INSERT



ROTATE



LOCK

Proper Fit Every Time

To ensure a proper fit, it is crucial to measure the stoma length using the AMT Balloon Stoma Measuring Device

Balloon Stoma Measuring Device	Order #	Packaged
Guidewire Compatible	A2-1804	10/Box
Non-Guidewire Compatible	A2-1704	10/Box

MiniONE® Balloon Button

AMT LENGTH (CM)	12F	14F	16F	18F	20F	24F
	ORDER NUMBERS					
	ENFit®					
0.8		A2-M1-51408-I				
1.0	A2-M1-51210-I	A2-M1-51410-I	A2-M1-51610-I	A2-M1-51810-I	A2-M1-52010-I	
1.2	A2-M1-51212-I	A2-M1-51412-I	A2-M1-51612-I	A2-M1-51812-I	A2-M1-52012-I	
1.5	A2-M1-51215-I	A2-M1-51415-I	A2-M1-51615-I	A2-M1-51815-I	A2-M1-52015-I	A2-M1-52415-I
1.7	A2-M1-51217-I	A2-M1-51417-I	A2-M1-51617-I	A2-M1-51817-I	A2-M1-52017-I	A2-M1-52417-I
2.0	A2-M1-51220-I	A2-M1-51420-I	A2-M1-51620-I	A2-M1-51820-I	A2-M1-52020-I	A2-M1-52420-I
2.3	A2-M1-51223-I	A2-M1-51423-I	A2-M1-51623-I	A2-M1-51823-I	A2-M1-52023-I	A2-M1-52423-I
2.5	A2-M1-51225-I	A2-M1-51425-I	A2-M1-51625-I	A2-M1-51825-I	A2-M1-52025-I	A2-M1-52425-I
2.7	A2-M1-51227-I	A2-M1-51427-I	A2-M1-51627-I	A2-M1-51827-I	A2-M1-52027-I	A2-M1-52427-I
3.0	A2-M1-51230-I	A2-M1-51430-I	A2-M1-51630-I	A2-M1-51830-I	A2-M1-52030-I	A2-M1-52430-I
3.5	A2-M1-51235-I	A2-M1-51435-I	A2-M1-51635-I	A2-M1-51835-I	A2-M1-52035-I	A2-M1-52435-I
4.0	A2-M1-51240-I	A2-M1-51440-I	A2-M1-51640-I	A2-M1-51840-I	A2-M1-52040-I	A2-M1-52440-I
4.4		A2-M1-51444-I	A2-M1-51644-I	A2-M1-51844-I	A2-M1-52044-I	A2-M1-52444-I
5.0		A2-M1-51450-I		A2-M1-51850-I		
5.5		A2-M1-51455-I				
6.0		A2-M1-51460-I		A2-M1-51860-I		
6.5		A2-M1-51465-I				
7.0		A2-M1-51470-I				
8.0		A2-M1-51480-I				
9.0		A2-M1-51490-I				
10.0		A2-M1-514100-I				

Feed Sets Included in Kit



Balloon Fill Volumes

French Size	Min	Rec.	Max
12F	2 ml	2.5 ml	3 ml
14F	3 ml	4 ml	5 ml
16F	4 ml	6 ml	8 ml
18F	6 ml	8 ml	10 ml
20F	7 ml	10 ml	15 ml
24F	7 ml	10 ml	15 ml




Recommended water fill volume is clearly printed on each MiniONE® Balloon Button fill valve. Consult your healthcare provider before adjusting fill volumes.

Complete Portfolio – Feed Sets

Adapter Options	Right Angle Connector Options			Straight Connector Options		
	2"	12"	24"	2"	12"	24"
ENFit® Y-Port		A2-81255-ISOSAF	A2-82455-ISOSAF			
ENFit® Single Port		A2-81221-ISOSAF	A2-82421-ISOSAF	A2-80211-ISOSAF	A2-81211-ISOSAF	A2-82411-ISOSAF
Clear ENFit® Y-Port XL	A2-80222-ISOSAF	A2-81222-ISOSAF	A2-82422-ISOSAF	A2-80212-ISOSAF	A2-81212-ISOSAF	

ENFit® Feed Sets included in kit

MiniONE® Balloon Button feed sets are compatible with Mic-Key® devices. Feed sets come packages as 10/box.

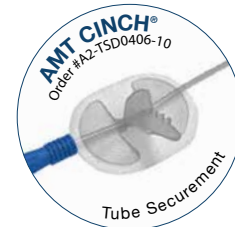

© 2020 Applied Medical Technology, Inc. C-3922-K I Patents: <http://www.AppliedMedical.net/Patents/> I Mic-Key® is a registered trademark of Avent, Inc. I ENFit® is a registered trademark of GEDSA I 1. Claims based on internal testing of 14F devices. Data on File. Content correct as of 12 April 2021

ENFit® Ready



An ISO 13485 Registered Company



Applied Medical Technology, Inc.
8006 Katherine Boulevard
Brecksville, OH 44141



AUS 1300 DEVICE (338 423)
customers@device.com.au
www.device.com.au

NZ 0508 DEVICE (338 423)
customers@device.co.nz
www.device.co.nz