





1

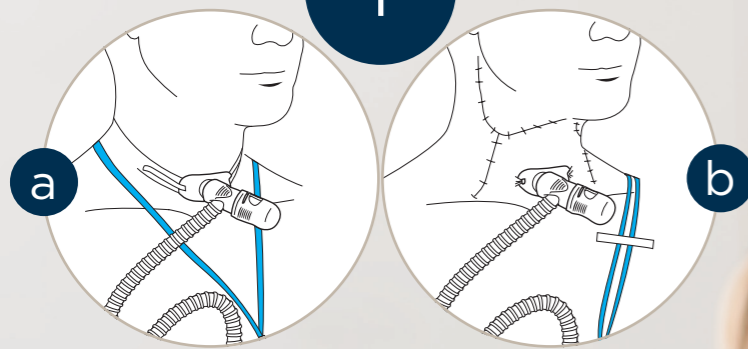

To ensure loading and movement on tracheostomy tube is kept to a minimum, make sure the lanyard is secured properly.

Lanyard position as shown in 1a is not suitable for patients with damaged skin.



3

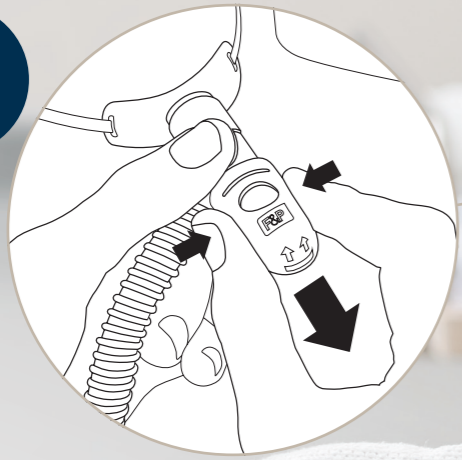


Orientate the sputum guard so that sputum drains downwards. The sputum guard can be removed if required.

Connect to the humidification system. Make sure all connections are secure.

2

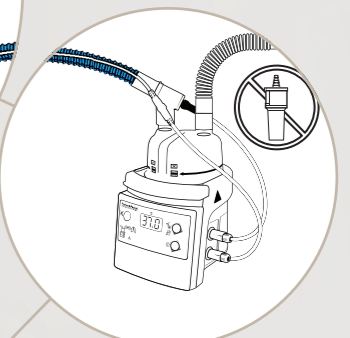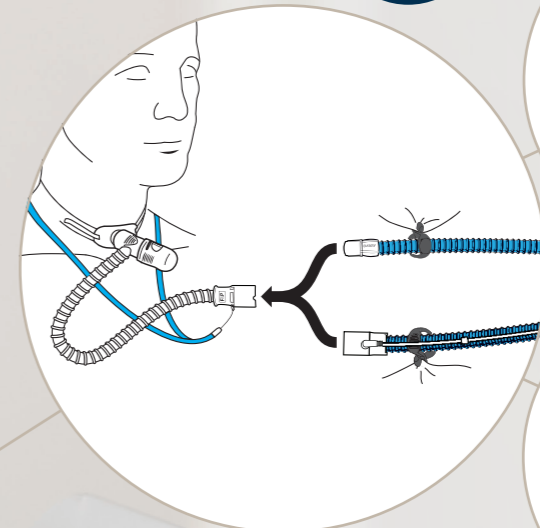

F&P 850 System

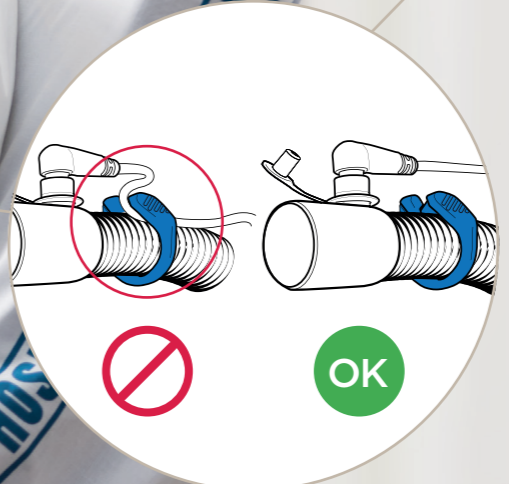

If using an MR850, attach the tubing clip to the breathing circuit but ensure probe cable is not crushed by tubing clip.

**i** IMPORTANT: Always refer to the user instructions supplied with the product for full set up instructions, warnings, cautions and contraindications.