



Lohmann & Rauscher

Vliwasorb[®] Pro

(Vlee-Va-Zorb Pro)

Highly conformable dressing
for highly exuding wounds



Vliwasorb® Pro


The advantages at a glance



Absorption


The spherical structure of the polymers provides greater surface area, giving Vliwasorb® Pro greater absorption capacity.

1. Absorption capacity
g/100cm²

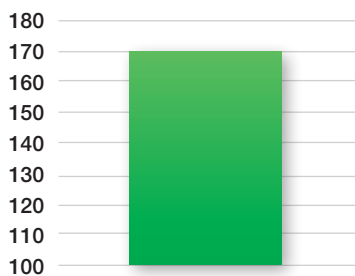

Vliwasorb® Pro

2. Absorption capacity
under pressure g/100cm²

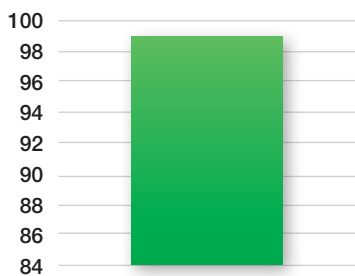

Vliwasorb® Pro

Measurements were taken using product samples in size 10 x 10 cm/12.5 x 12.5 cm.
The measurements refer to values from one lot and can only be evaluated accordingly.

1 Absorption capacity: PV-QMS-A-011 based on DIN EN ISO 13726

2 Absorption capacity under pressure: internal method description MB 44 based on SMTL TM-414



Comfort and conformability

■ Ergonomic shape, flexible design and softer absorbent core

- easy to retain
- comfortable for patients to wear



Effective exudate management

■ Very high absorbency and retention capacity

- protects against maceration - improving peri-wound skin
- longer dressing change intervals - saving time and money, with less disturbance of the wound
- effective under compression - a combined approach can be taken to optimize healing potential



Locks away harmful proteases and bacteria

■ Proteases and microbes are bound in the superabsorbent polymer core

- reduces the microbial burden
- reduces odours



Protection and safety


■ Blue clothing protection layer and smooth wound contact layer

- atraumatic dressing changes
- protects the patient's clothing and bed linen

In combination these advantages lead to improved patient quality of life.


The structure of Vliwasorb® Pro

Non-wound side

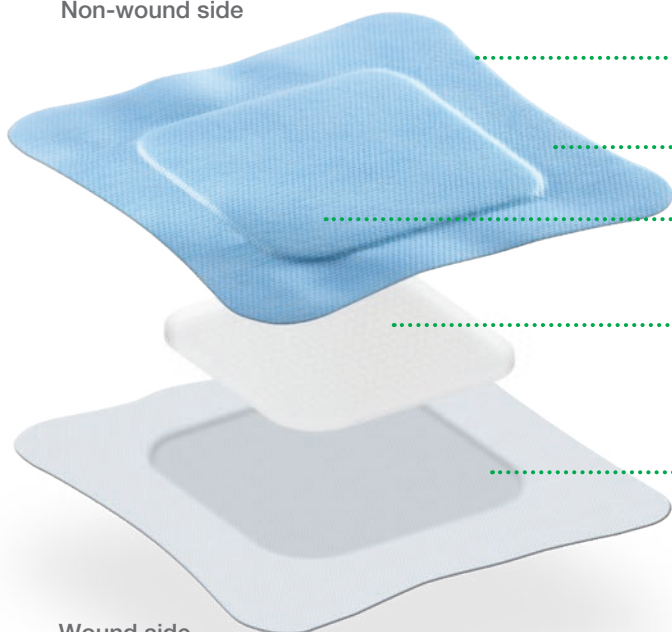


Wound side

Rounded edges and ergonomic shape **for good conformability**

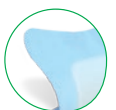


Wide edges **for high wearing comfort**

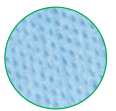

Blue clothing protection layer **protects clothes and bed linen**



Superabsorbent core with distribution layer **for effective exudate management and flexibility**

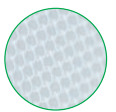

Smooth wound contact layer **for atraumatic dressing changes**



Did you know?

Vliwasorb® Pro may be used as a primary wound dressing - its integrated wound contact layer negates the need to use an additional contact layer underneath, enhancing cost effectiveness.

Indications

Superficial wounds with moderate to very high levels of exudate, including:



Chronic wounds e.g. leg ulcers, pressure ulcers, diabetic ulcers, oncological wounds




Acute wounds e.g. partial thickness burns, postoperative wounds, trauma wounds, skin graft donor sites

Ensure appropriate retention of Vliwasorb® Pro to secure the dressing against the wound. If compression (e.g. Rosidal®) is not appropriate then an effective retention bandage should be used.

Evidence

In a 171 patient study, where Vliwasorb® Pro was used for a minimum of 3 dressing changes, clinicians were asked for feedback on a number of criteria. The findings were...

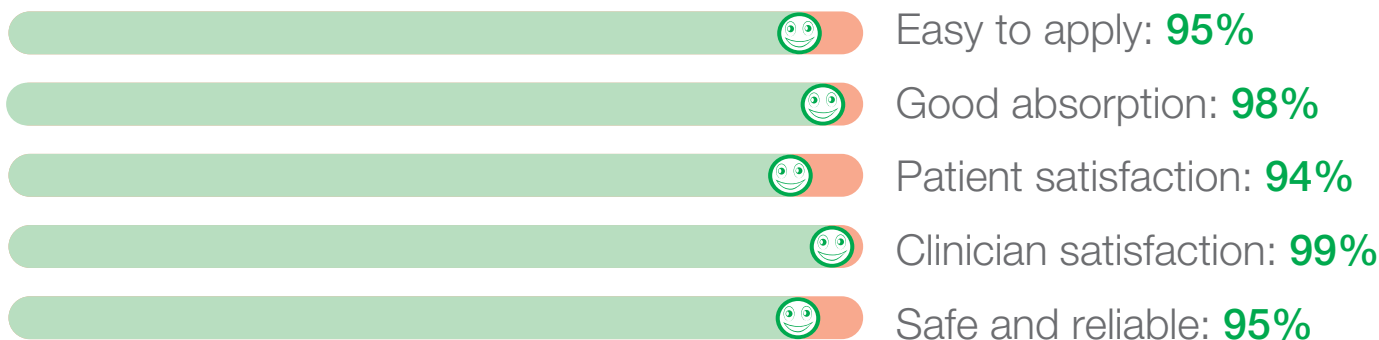

Efstathios et al. (2017) A comfortable superabsorbent dressing for management of patients with moderate-to-highly exuding wounds. Poster presentation at Wounds UK Conference, Harrogate.

Please ask for our evidence summary for more detail and further studies

Ordering information

Vliwasorb® Pro

sterile, individually sealed

Size (cm)	Wound pad (cm)	REF	Carton / Box
10 x 10	6 x 6	32 640	10 / 300
12.5 x 12.5	8.5 x 8.5	32 641	10 / 300
12.5 x 22.5	8.5 x 18.5	32 642	10 / 350
22 x 22	18 x 18	32 643	10 / 260
22 x 32	18 x 28	32 644	10 / 90

Note: Please check the size of the wound pad before placing your order.

For any queries please contact your local Lohmann & Rauscher (L&R) Sales Representative or L&R Customer Service on:

Australia: 1300 572 869 New Zealand: 0800 572 869 Email: info@au.LRMED.com

Lohmann & Rauscher Pty Ltd · 321 Ferntree Gully Road · Mount Waverley, Victoria 3149 · Australia
Lohmann & Rauscher Limited · 21 Queen Street · Auckland 1010 · New Zealand

