

WHITE SOFT PARAFFIN B.P. STERILE

Composition	White Soft Paraffin B.P.
Use	A lubricant for catheters and surgical instruments. An aseptic occlusive dressing for wounds.
Contraindications	Known hypersensitivity to petroleum based products.
Directions for use	Apply as necessary to instruments where lubrication is required. Apply to wounds as required. If irritation occurs, stop use immediately and seek medical advice.
Precautions	Not for ophthalmic use. Single use only.
Adverse Effects	Adverse effects of soft paraffin are rare when used in topical preparations, but sensitivity reactions and acne have been reported following topical use.
Storage	Store below 30°C.
Poison Schedule	Not scheduled.
Presentation	ARTG No. 155358 (lubricant), ARTG No 155368 (occlusive wound dressing): White Soft Paraffin BP Sterile is supplied in single use 5g and 10g tubes. Each tube contains White Soft Paraffin BP sterilised by gamma irradiation. The contents of the tube are sterile until opened.

Code	Product	Size
WHI01830F	WHITE SOFT PARAFFIN BP STERILE (tube)	5g
WHI01657F	WHITE SOFT PARAFFIN BP STERILE (tube)	10g

Contact Point

Distributed by Perrigo Australia
 Orion Laboratories Pty. Ltd., trading as Perrigo Australia
 ABN 56 009 293 136
 25-29 Delawney Street, Balcatta, Western Australia 6021 AUSTRALIA
Telephone (all hours): +618 9441 7800
 FREE PHONE: 1800 805 546 FREE FAX: 1800 004 110
 EMAIL: customerservice@perrigo.com.au; WEBSITE: www.perrigo.com.au
 ORION® is a registered trademark of Orion Laboratories Pty Ltd

END OF USER GUIDE