


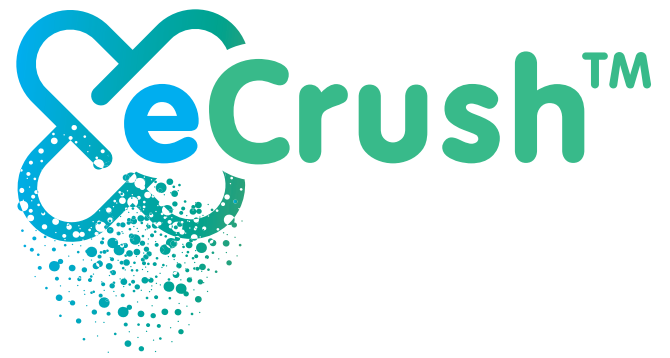

Electronic tablet crusher

From pill to powder in seconds

- Quick, easy to operate, electronic tablet crusher
- Crushes tablets into a fine, consistent powder in 22 seconds
- Avoids physical strain from repetitive manual crushing
- Easy to clean, medical grade device
- Rechargeable battery with LED battery level indicator screen
- 3 button operation: power, open and crush
- Sturdy plastic single-use cups ensure minimal dosage loss and no cross-contamination
- 2 year warranty. Re-use of the cups will void the product warranty
- eCrush packaging includes: eCrush device, AC/DC adapter and power cord, re-chargeable Li-Ion battery, screwdriver, cup holder, and 100 crushing cups

**Some medications should not be crushed, always check with a pharmacist prior to crushing a medication*




Australian/NZ Distributor:

evomed
innovative healthcare products

For more information call 0409 218 996
or email: customerservice@evomed.com.au


www.tabletcrushers.com.au

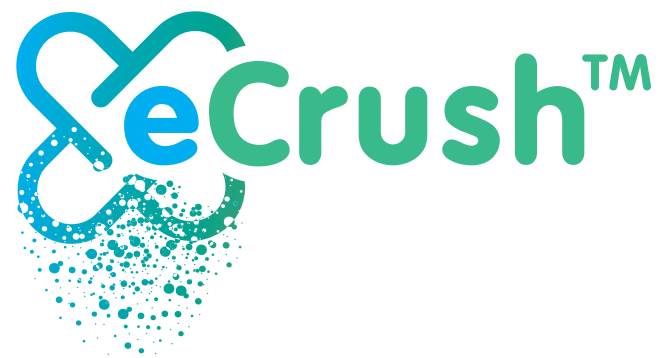


Quick start guide



1. Place first disposable cup into the blue cup holder

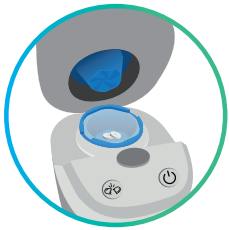

2. Place medication in the cup (max. 4g)

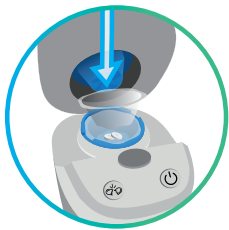

3. Place second disposable cup on top



4. Close lid



5. Push on-button then push crush-button

Technical specifications	
Input (Power)	AC/DC adapter, 100-240V ~, 50/60Hz
Battery	Li-ion 14.8V 4400mAh Approx 500 crushing cycles/charge Approx 3-4hrs to fully charge
Crushing cycle time	22 seconds
Approximate crushing cycles	100,000 times
Crushing cup material	Food grade polypropylene that is recyclable
Crushing cup volume	30ml
Max. weight of pills per crush	≤ 4g
Dimensions (mm)	191.5 (L) x 147 (W) x 194 (H)
Weight	3.82kg
Cleaning	Wipe with a dry/slightly damp cloth

Australian/NZ Distributor:

evomed
innovative healthcare products

For more information call 0409 218 996
or email: customerservice@evomed.com.au

www.tabletcrushers.com.au